

## BACK APPLICATOR QUICK SETUP GUIDE



1.) AFTER SECURING YOUR HORSE, PLACE THE CONTROLLER IN AN AREA AWAY FROM THE HORSE.



2.) INSERT THE COILS INTO THE BACK APPLICATOR POCKETS. DOTS SHOULD ALWAYS FACE OUT, AWAY FROM THE INJURED AREA.



3.) SECURE THE APPLICATOR USING STRAPS AND ROUTE THE CABLES TO KEEP THEM OFF THE GROUND.



THE CYTOWAVE BACK APPLICATOR COMES WITH NUMEROUS DESIGNED ATTACHMENT POINTS. THIS ALLOWS FOR MAXIMUM FLEXIBILITY FOR EACH HORSE OWNER TO SECURE THE BLANKET SAFELY.





5.) CONNECT THE APPLICATOR POWER CABLE TO THE CONTROLLER POWER CABLE. NEVER RUN THE CABLE ALONG THE GROUND WHERE A HORSE CAN EASILY SEVER IT BY STEPPING ON IT.



6. MAKE SURE THAT YOUR HORSE IS SECURE BEFORE BEGINNING TREATMENT. SEE THE CONTROLLER OPERATIONAL GUIDE FOR TREATMENT OPTIONS.



COILS CAN BE PLACED IN THE BACK APPLICATOR BEFORE PLACING ON THE HORSE OR AFTER. FIRST TIME USERS MAY WANT TO PLACE THE BACK APPLICATOR ON THE HORSE WITHOUT THE COILS TO GAUGE A HORSE'S COMFORT LEVEL WITH THE APPLICATOR.

AVOID YOUR HORSE CHEWING ON THE CABLES - THERE IS NO RISK OF SHOCK BUT THEY CAN BE EXPENSIVE TO REPLACE.

CYTOWAVE STRONGLY RECOMMENDS THAT CLIENTS CONSULT WITH A VETERINARIAN TO DIAGNOSE THE CONDITION, MONITOR TREATMENT AND RECOMMEND A REHABILITATION REGIMEN. YOUR HORSE WILL BECOME SOUND BEFORE IT IS READY TO RESUME FULL ACTIVITIES. ***DO NOT PUSH YOUR HORSE TOO FAR, TOO SOON.***

Questions? Call us at 844-298-9283

