

## HOCK APPLICATOR QUICK SETUP




1.) OPEN BOOT BY DETACHING THE VELCRO STRAPS

2.) STARTING AT THE TOP, WRAP THE BOOT AROUND THE HOCK.

3.) THE RING SIDE OF THE BOOT SHOULD PASS UNDER THE STRAP SIDE OF THE BOOT (NOTE THE BLUE ARROW).




4.) THE TOP ELASTIC STRAP SHOULD RUN THROUGH THE STRAP LOOP AT THE BACK OF THE BOOT.

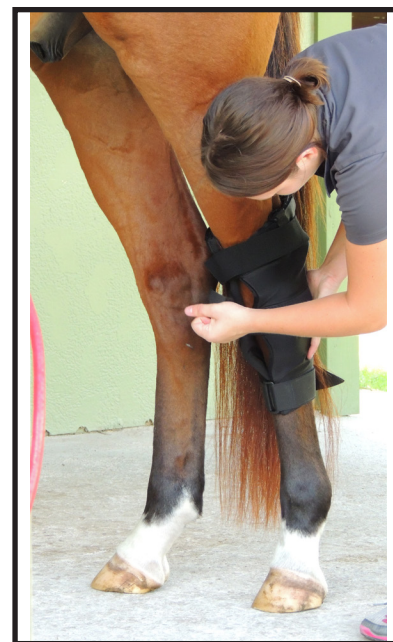

5.) THREAD THE REMAINING STRAPS THROUGH THEIR RESPECTIVE LOOPS AND TIGHTEN THE STRAPS SO THE BOOT IS HELD FIRMLY IN PLACE.

## ATTACHING THE COIL

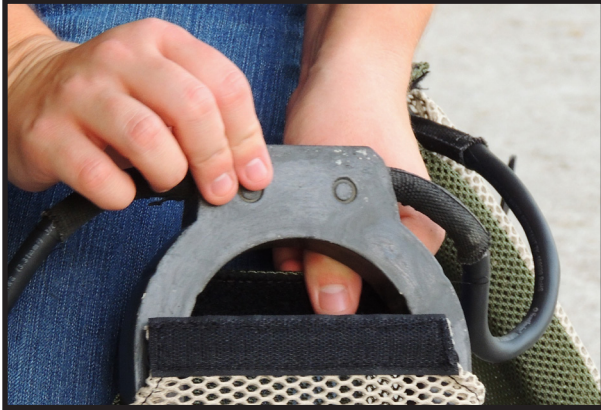

1.) INSERT THE COILS INTO THE COIL POCKETS. THE FLAT SIDE OF THE COIL RESTS ON THE VELCRO SIDE OF THE POCKET AND THE DIMPLES ON THE COIL FACE OUTWARD.



3.) ADJUST THE COILS TO ALIGN WITH THE INJURED AREA.

2.) SNAP THE COIL CLIPS TO THE BOOT COIL CLIPS STARTING AT THE TOP. THE COIL CLIPS SHOULD BE PROPERLY ALIGNED WITH THE BOOT CLIPS AND SNAPPED IN PLACE.



4.) CONNECT THE APPLICATOR POWER CABLE TO THE CONTROLLER POWER CABLE AND REFER TO THE CONTROLLER USER GUIDE TO SELECT THE APPROPRIATE TREATMENT.



AVOID YOUR HORSE CHEWING ON THE CABLES - THERE IS NO RISK OF SHOCK BUT THEY CAN BE EXPENSIVE TO REPLACE. WHEN STORING, TRANSPORTING OR SHIPPING OUR BOOTS, PLEASE MAKE SURE THEY ARE RESTING FLAT OR IN A SOFT/GENTLE CURVE TO AVOID CRACKING.

CYTOWAVE STRONGLY RECOMMENDS THAT CLIENTS CONSULT WITH A VETERINARIAN TO DIAGNOSE THE CONDITION, MONITOR TREATMENT AND RECOMMEND A REHABILITATION REGIMEN. YOUR HORSE WILL BECOME SOUND BEFORE IT IS READY TO RESUME FULL ACTIVITIES. **DO NOT PUSH YOUR HORSE TOO FAR TOO SOON.**