

LEG APPLICATOR QUICK SETUP GUIDE v3



1) ALIGN THE BOOT WITH THE STRAPS FACING TOWARD THE FRONT. THE STRAP SIDE OF THE BOOT SHOULD ALSO FACE OUTSIDE, AWAY FROM THE HORSE.



2.) STARTING AT THE TOP, WRAP THE BOOT AROUND THE LEG (YELLOW ARROW).



3.) STARTING FROM THE TOP, LOOP THE FIRST STRAP THROUGH THE STRAP RING AND WRAP BACK OUT TOWARD THE OUTSIDE - THE RINGED SIDE WILL FOLD UNDER THE STRAP SIDE OF THE BOOT.



4.) WORK DOWN, TIGHTENING THE STRAP FOR UNIFORM PRESSURE ALONG THE LENGTH OF THE LEG. THE BOOT SHOULD BE HELD FIRMLY IN PLACE.



ATTACHING THE COIL



1.) POSITION THE BOOT CLIPS SO THEY ARE ACCESSIBLE.

2.) PLACE THE COIL ALONG THE BOOT WITH THE DIMPLES FACING OUTWARD AND SNAP THE COIL CLIPS TO THE BOOT CLIPS, STARTING AT THE TOP.



3.) CINCH THE COIL STRAPS TIGHT. THE COILS SHOULD BE PARALLEL TO EACH OTHER AND REST FLAT AGAINST THE BOOT.



4.) CONNECT THE APPLICATOR POWER CABLE TO THE CONTROLLER POWER CABLE. NEVER RUN THE CABLE ALONG THE GROUND WHERE A HORSE CAN EASILY SEVERE IT BY STEPPING ON IT. INSTEAD, PLACE IT OUTSIDE OF THE STALL OR HANG NEARBY.



CARE Instructions

Washing - secure velcro, remove padding
use microbacterial soap
gentle or hand wash, hang to dry



AVOID YOUR HORSE CHEWING ON THE CABLES - THERE IS NO RISK OF SHOCK BUT THEY CAN BE EXPENSIVE TO REPLACE. WHEN STORING, TRANSPORTING OR SHIPPING OUR BOOTS, PLEASE MAKE SURE THEY ARE RESTING FLAT OR IN A SOFT/GENTLE CURVE TO AVOID CRACKING.

CYTOWAVE STRONGLY RECOMMENDS THAT CLIENTS CONSULT WITH A VETERINARIAN TO DIAGNOSE THE CONDITION, MONITOR TREATMENT AND RECOMMEND A REHABILITATION REGIMEN. YOUR HORSE WILL BECOME SOUND BEFORE IT IS READY TO RESUME FULL ACTIVITIES. **DO NOT PUSH YOUR HORSE TOO FAR TOO SOON.**

CYTOWAVE 1201 JUPITER PARK DRIVE, JUPITER, FLORIDA 33458 844-298-9283 CYTOWAVE.COM